

THANK YOU!

For purchasing the Kneevoice Cartilage Auscultation System.



Follow these simple instructions in numerical order.


 **Note:** This guide does not replace the official Instructions for Use (IFU). Please consult the IFU for complete safety warnings.

 **Support:** <https://www.kneevoice.com/contact-us/>




1. Packing List

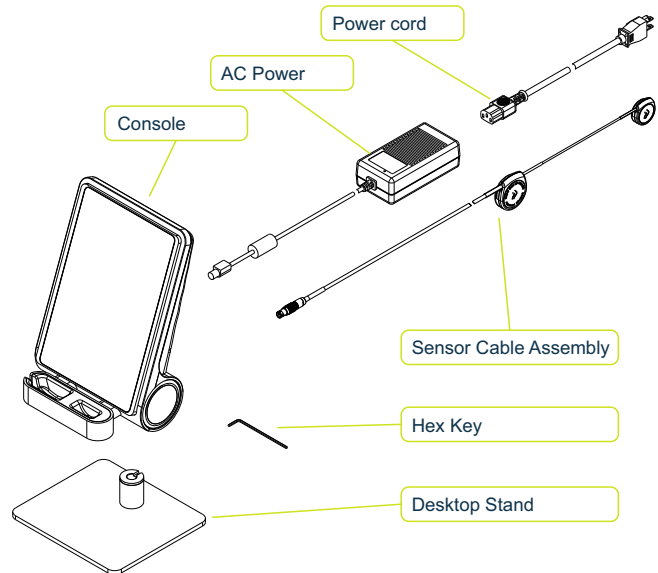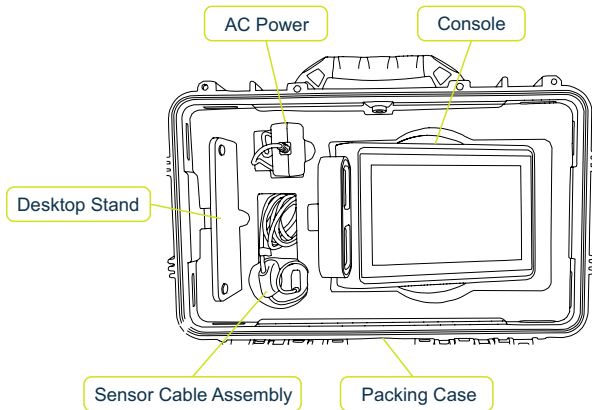

 Keep the packing case for possible future transport.


2. Console Mounting:

Insert the console's mounting post into the base. Tighten the set screw on the back with the included hex key.

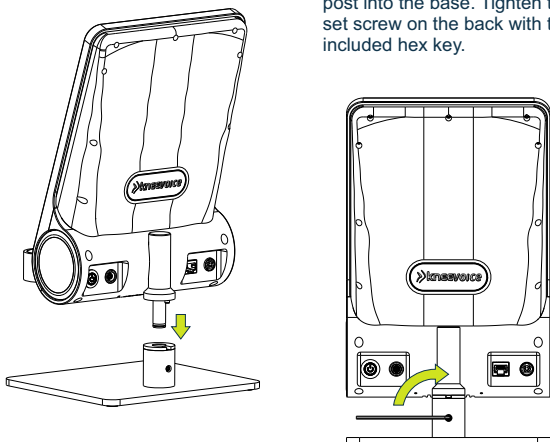



 Keep the hex key for possible future disassemble.

3. Power Connection:

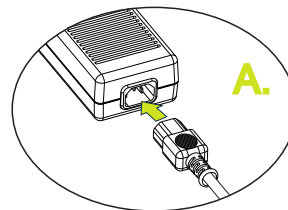

A. Connect power cord to the power supply.



B. Connect the power supply cable to the designated port on the back of the console.

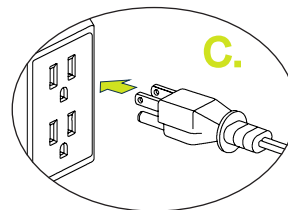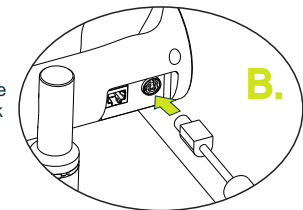

C. Plug the power cord into a 110V outlet.

4. Power On & Sensor Connection

A. Press the Power button and enter your 4-digit PIN (or follow the on-screen instructions to create one if this is the initial setup).

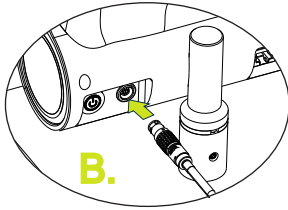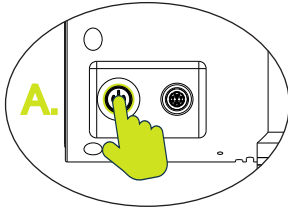

B. Connect the sensor cable to the console.


5. Patient Preparation:

A. Have the patient sit upright on an exam table or high chair.



i Important: Feet must NOT touch the floor

B. Select NEW PATIENT (enter details) or EXISTING PATIENT (choose from list).

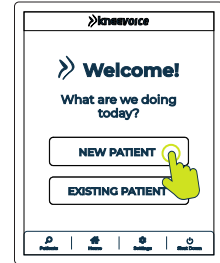


C. Select the leg, set the Pain level (0-10), and tap START NEW EXAM.



6. Sensor Placement

First-time users: After tapping START NEW EXAM, tap NEXT (do not skip) to view the on-screen tutorial for patch application and sensor placement.

- A. Clean skin with 70% Isopropyl alcohol.
- B. Apply new adhesive patches
- C. Place Audio Sensor on kneecap
- D. Place Position Sensor on the RIGHT side of the lower leg
- E. Ensure cable routes to the LEFT

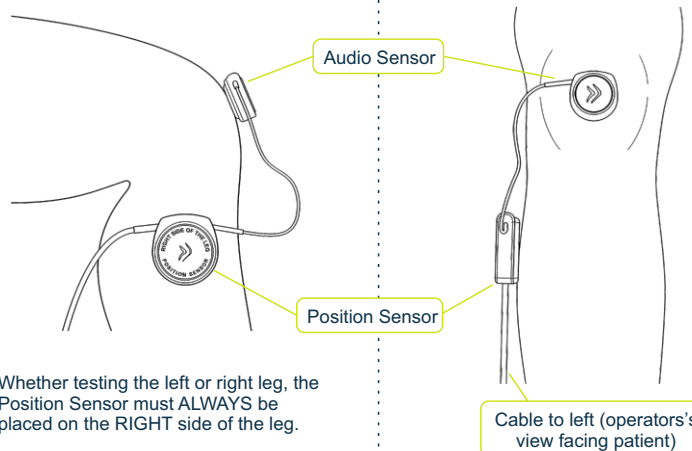


i Whether testing the left or right leg, the Position Sensor must ALWAYS be placed on the RIGHT side of the leg.

7. Calibration & Exam

A. **Calibration:** Touch START TRAINING and have patient extend leg twice (0°-90°)




B. **Exam:** Then, select START EXAM. Patient must complete 8 extensions following the on-screen cadence.

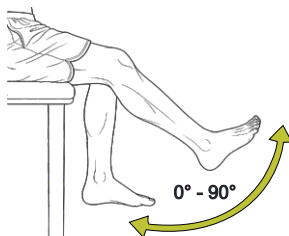


i Important: The patient must remain in total silence during the exam.

8. Completion & Cleaning

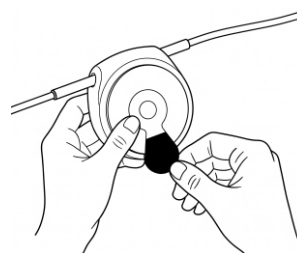

A. Remove patches and dispose of as biohazard waste.

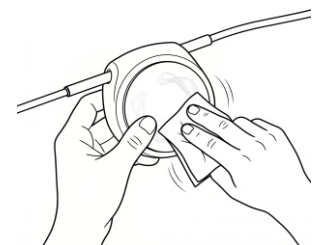

B. Manually wipe sensors and touchscreen with 70% alcohol wipes.

⚠ Do not immerse in liquids.